# podifer40

RHEUMA-EINLAGE - RHEUMATIC INSOLE - SEMELLE RHUMATOÏDE  
РВМАТИЧЕСКИЙ СУПИНАТОР - PLANTARE REUMATICO

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Rheumatism relief insoles for normal activity. The insoles are made with different foams (including a layer of EVA foam) that eliminate any pressure points which helps to alleviate joint pain, especially joint pain experienced during the early stages of rheumatism. Walking becomes easier and more natural with less strain on the joints and the delayed onset of any deformity. The special fabric covering woven with silver thread is antibacterial with odour control.

**INDICATIONS:** Hammer toe, in the initial stages • Bunions, in the initial stages • Arthralgia • Metatarsalgia • Flatfoot or high arched foot.

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Top layer of fabric made with woven silver threads with natural anti-bacterial and odour-inhibiting effect.

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The special EVA layer automatically shapes itself according to the pressure it is exposed to, filling out the areas exposed to lower pressure, thus ensuring increased wearing comfort.

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The middle visco-elastic layer with form memory effect, in an optimised thickness and with damping function.

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Lower shell made of rigid PU for the stabilisation and support of the foot in pronation and supination.

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The bottom layer is made of PU and acts as a protective layer for the layers above, providing protection against wear and deformation. It also makes it easier to position the insole in footwear.

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## ENG. PUTTING ON THE APPLIANCE

### FIRST APPLICATION

1. Remove the original insole from the patient's shoe.
2. Place the insole in the shoe, check that it is correctly positioned, and ask the patient to try it on.
3. Alterations may be made to the insole to suit individual requirements (it is advisable for the patient to wear the insole for a few hours before making any alterations). The front can be shaped with scissors (use the insole that was previously in the patient's shoe for reference). The arched detailing supporting the longitudinal arch can be filed down a little at a time, as can the outer edge of the rear part of the insole.
4. We recommend removing the insole from the shoe after wear so that it can air.

### SUBSEQUENT WEAR

1. Place the insole in the shoe and check that it is correctly positioned.
2. We recommend removing the insole from the shoe after wear so that it can dry.

### CONTRAINDICATIONS

Currently no known

### DECLARATION OF CONFORMITY

As the manufacturer, ORTHOSERVICE AG declares with sole responsibility that this product is a Class I medical product and has been manufactured in accordance with the basic requirements of the directives 93/42/EEC and 2007/47/EEC. These operating instructions were created in accordance with the specification of the above-mentioned directives. They aim to ensure the safe and correct use of the medical product.

### MAINTENANCE

#### Washing instructions:

Wash by hand up to 30°C with neutral soap; let it dry away from heat sources. Do not dispose of the product or any of its components into the environment.

### PRECAUTIONS

It is advised that the pressure exerted by the product does not act on parts of the body suffering from wounds, swelling or lumps. It is advisable to wear the product over a garment, avoiding direct contact with the skin. If in doubt about how to apply, contact an orthopaedic technician. The device should not be worn in the vicinity of open flames.

### WARNINGS

The product may only be used in the event of medical indication or when prescribed by a physician and must be applied by an orthopaedic technician, the competent authority figure for the application as well as for information on the safe use, according to individual needs. To ensure the effectiveness, tolerability, and correct operation, it is necessary that the device is put on with the utmost care. Any changes to the structure or adjustment of the device must be prescribed by a physician and performed by an orthopaedic technician. Under no circumstances should the adjustment made by the physician/orthopaedic technician be altered. Use of the product is recommended for a single patient. With hypersensitive patients, direct contact with skin may cause redness or irritation. In the case of onset of pain, swelling, lumps or any unusual reaction, immediately contact your physician.

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